





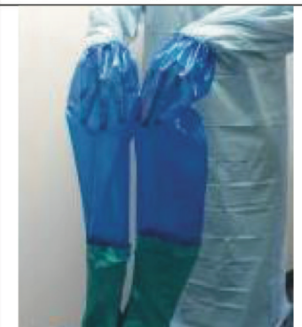
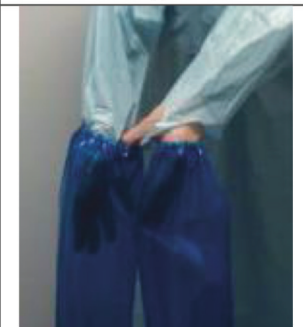



# Long Sleeve Glove

<b>Brand Name of Product</b>	Decontam Long Sleeve Gloves
<b>Generic Name of Product</b>	Sleeve Gloves
<b>Product Code Number(s)</b>	41380, 41480, 41580, 41680
<b>Intended Use:</b>	For the safe handling of contaminated items in the decontamination area of a sterile processing department. Features long VR™ sleeves connected to gloves for more protection than normal gloves.
<b>Range of Applications for Product</b>	Handling potentially dangerous contaminated medical devices.
<b>Key specifications of product</b>	<ul style="list-style-type: none"> <li>• Fusion of 15 mil nitrile latex free lined glove and 4mil vinyl protective sleeve</li> <li>• Available in various lengths and sizes</li> <li>• Comfortable, flexible and lightweight with elastic cuffing.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

Instructions for Using Product	
<b>Description of Use (s)</b>	
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures):</b>	<p style="text-align: center;"><b>DONNING</b></p> <div style="display: flex; justify-content: space-around;">    </div> <p style="text-align: center;"><b>DOFFING</b></p> <div style="display: flex; justify-content: space-around;">    </div> <div style="display: flex; justify-content: space-around;">    </div>
<b>Steps for Use of Product</b>	

	<p><b>Donning:</b></p> <ol style="list-style-type: none"> <li>1. Begin pulling on glove fingers and thumb until they are in the palm of the glove.</li> <li>2. Pull the fingers of the opposite glove so that both hands are into the palm or cuff area of the gloves.</li> <li>3. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve – repeat with the opposite hand.</li> </ol> <p><b>Doffing (If reuse is intended clean prior to removal):</b></p> <ol style="list-style-type: none"> <li>1. Begin pulling on glove fingers and thumb until they are in the palm of the glove.</li> <li>2. Pull the fingers of the opposite glove so that both hands are in the palm or cuff area of the gloves.</li> <li>3. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve – repeat with the opposite hand.</li> <li>4. The sleeves are ready for removal once both hands reach the elastic cuffs.</li> <li>5. Grasp the inside of the opposite sleeve with the thumb and fore finger to remove both sleeve gloves.</li> <li>6. Sleeve Gloves are ready to hang dry or be properly disposed. If cleaning for reuse, do so prior to doffing.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Carefully inspect the Sleeve Gloves for punctures and cuts.</li> <li>• Sleeve Gloves may be cleaned and rinsed while being worn.</li> <li>• Scrub Sleeve Gloves thoroughly with appropriate cleaning agents for proper cleaning and disinfecting. Rinse and hang dry.</li> <li>• A list of recommended cleaning and disinfecting cleaning and disinfecting agents is available on our website.</li> </ul>
<b>Disposal</b>	Treat used gloves as biohazard and dispose of in accordance with facility policy for contaminated items when disposing.

<b>Reprocessing Instructions</b>	
<b>Point of use:</b>	
<b>Preparation for decontamination:</b>	
<b>Disassembly Instructions:</b>	
<b>Cleaning – Manual:</b>	Clean with detergent with compatible chemicals, hot water and rinse.
<b>Cleaning – Automated:</b>	
<b>Disinfection:</b>	
<b>Drying:</b>	
<b>Maintenance, inspection, and testing:</b>	
<b>Reassembly Instructions:</b>	
<b>Packaging:</b>	
<b>Sterilization:</b>	
<b>Storage:</b>	
<b>Additional Information:</b>	When re-using the gloves, it is recommended to dispose after use that day.