

Famos Bowie-Dick Test Pack

Cat. No. 15.100.039 Air removal test for pre-vacuum sterilizers


Instruction for use:

" This test should be performed on daily or with the beginning of each shift"

- 1-Pre-heat the sterilizer by running one complete cycle.
- 2-Place the test pack horizontally on a rack over the chamber drain within the preheated empty sterilizer.
- 3-Run the air removal cycle with 134°C -137°C for up to 3.5 minutes, after this the test will be removed directly, open and examined the test in regard to the color change.
- 4-Fill out the information as indicated on the back of the card.




Unprocessed TEST
Original color (Yellow)

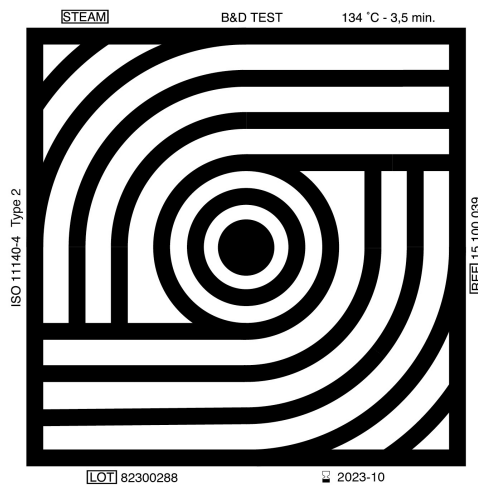

"Passed" processed TEST
TEST Color (Black) ✓

Failure results:

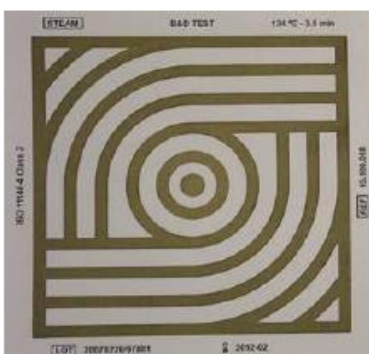

Fail Temperature too low X



Fail Insufficient vacuum/penetration X



Fail Small air/gas collection X

Famos Medizintechnik Vertriebs GmbH,

Ro-bert-Bosch-Straße 3 T : +49 - (0)2834 6704,
47638, Straelen Germany F : +49 - (0)2834 8571,
E : sales@famos-med.de,
www.famos-medical.com